



Start Up and Warranty Registration Form

Source Equipment / Vacuum Systems

Date of Start up: _____ **Project Number:** _____

Pre-start and initial start up procedures must be performed in accordance with instructions provided in the Amico Operation and Maintenance Manual

Start Up Representative	End User
Name: _____	Name: _____
Address: _____	Address: _____
Representative: _____	Representative: _____
Phone Number: _____	Phone Number: _____
Fax Number: _____	Fax Number: _____

Unit Information (located on the side of the control panel)

Model #: _____	Project #: _____	Ship Date: _____
Voltage: _____	System FLA: _____	Drawing #: _____ Rev.: _____

Installation (see manual for installation requirements)

Altitude of Installation: _____

Ambient Temperature: _____ Highest Expected Ambient Temperature: _____

Is system mounted level: Yes No Ventilation: Adequate Inadequate

Location/Room: _____ Primary Circuit Breaker: Single Multi-Feed

Primary Supply Voltage: _____ Primary Supply Amperage: _____

Service Access around equipment – Minimum 24" required: Yes No

Facility vacuum piping size: _____ Facility exhaust piping size: _____

Comments: _____

Pre-Start Checks (see manual for pre-start checks)

Pump Motor Nameplate Amps: _____ Overload Relay Setting: _____

Customer connections complete: Vacuum Exhaust Electrical

Exhaust drip legs installed on all vacuum pumps: Yes No

All shut off valves correctly positioned: _____ Oil level checked (if app – all pumps): _____

Direction of rotation checked/corrected on all pumps: Yes No

This form must be completed and returned to Amico Source Corporation within five days to register the unit for warranty.
You can find this form online at www.amico.com



Start Up and Warranty Registration Form Source Equipment / Vacuum Systems

Project Number: _____

Operational Checks (see manual for start up procedures)

#1 Motor Volts:	L1-L2:	L2-L3:	L1-L3:
#2 Motor Volts:	L1-L2:	L2-L3:	L1-L3:
#3 Motor Volts:	L1-L2:	L2-L3:	L1-L3:
#4 Motor Volts:	L1-L2:	L2-L3:	L1-L3:
#5 Motor Volts:	L1-L2:	L2-L3:	L1-L3:
#6 Motor Volts:	L1-L2:	L2-L3:	L1-L3:
#1 Motor Amps:	L1:	L2:	L3:
#2 Motor Amps:	L1:	L2:	L3:
#3 Motor Amps:	L1:	L2:	L3:
#4 Motor Amps:	L1:	L2:	L3:
#5 Motor Amps:	L1:	L2:	L3:
#6 Motor Amps:	L1:	L2:	L3:
Vacuum switch settings:	Lead:	2nd:	3rd:
	4th:	5th:	6th:

Alternation verified: Yes No Noise & Vibration OK: Yes No

Hand-Off-Auto switch checked for proper operation: _____

Hand-Off-Auto switch left in which position (Auto recommended): _____

Relief valve setting (if applicable):	Pump 1:	Pump 2:	Pump 3:
	Pump 4:	Pump 5:	Pump 6:

Maximum vacuum obtained with system isolation valve closed: _____

Discharge temperature for all pumps normal: Yes No

Alarms checked: High Temp Lights: Yes High Temp Shutdown Function: Yes

Lag Pump Run: Yes

Oil level rechecked and topped off after running - all pumps (if applicable): Yes

System Status

Online Offline If Offline, for how long? _____

Comments:

We the undersigned, having observed the start up of this equipment, certify that the information given is true and correct to the best of our knowledge.

Amico Representative: _____ Date: _____

End User Representative: _____ Date: _____